



# Pressure therapy 8 sectors

## LEG - 8 SECTORS



The machine has 20 programs that vary according to the combinations of the different pressures, times and alternations of inflation of sectors to simulate different types of massage. This is an ad hoc method to make the machine easy to use even by inexperienced operators.

The device improves lymphatic circulation and promotes the draining of fluids and waste products. By presetting parameters such as pressure, time of inflation/deflation and duration of the treatment, it is possible to select the type of treatment that targets the blemish in question. The operator can modify the pressure on the different sectors during the treatment without compromising it in any way. Of course,

the increase in pressure in the different sectors must be adapted to the client's needs, and can reach a maximum value calculated according to the patient's minimum (diastolic) blood pressure (5% less than the minimum).

#### Actions:

- Reduces visible cellulite
- Reduces localised and diffused fat on the legs, gluteus and abdomen
- Reduces oedemas and lymphoedemas
- Improves peripheral micro-circulation
- Reduces fatigue on the legs

**PRESSURE THERAPY - 8 SECTORS** Device to apply pressure massage, designed for the market of professional aesthetic treatments. The device uses a series of information provided by the client to create personalised programs specific for the type of problem in question, without the potential risk of applying treatments that are useless or hazardous for the client's well-being. It uses the combined action of the different sectors to reproduce the various types of lymphatic drainage manual massage, such as Vodder or toning massage against cellulite of different stages.



**TECHNICAL DATA SHEET**

<b>Input voltage</b>		230 V - 50 Hz
<b>Absorbed power</b>		250 VA
<b>User interface/display</b>	<b>MOD. 1</b>	3,7" b&w - 240x128 px
	<b>MOD. 2</b>	7" Wide Screen - 800x480 px
	<b>MOD. 3</b>	40x2 Alphanumeric
<b>Operation pressure</b>		1-100 mmHg
<b>Output channels</b>		8
<b>Type of emission</b>		Continuous
<b>Programs</b>		20
<b>Size of device</b>		50x40x20 cm
<b>Weight</b>		11 Kg
<b>Safety class</b>		1 / BF

# Accessories

---

## PRESSURE THERAPY 8 SECTORS

### CORSETRY

1. 2 BOOTS WITH TWO INDEPENDENT SECTORS,  
2 LEGS WITH 4 INDEPENDENT SECTORS,  
2 ADJUSTABLE BAND WITH 2 INDEPENDENT  
SECTORS EACH, FOR THE TREATMENT OF  
FLANKS AND BUTTOCKS
2. LEG AREA CONNECTION CABLE
3. SAFETY BUTTON

### OPTIONAL

4. BLOOD PRESSURE METER WITH CABLE  
CRYO/WARM OVER HANDPIECE

